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MELAcontrol is a sterilization indicator and batch control system that consists of a helix test body and an indicator strip.

MELAcontrol is only designed for testing Class B autoclaves, i.e. autoclaves which work with the fractionated (pulsed) vacuum method.

1. Unscrew the cap from the helix test body.

2. Fold an indicator strip in the middle (text on the inside).
3. Insert the middle fold part of the indicator strip first into the closure cap of the helix test body.
4. Insert the indicator strip as far as it will go into the cap.
5. Screw the closure cap back into the helix test body as far as it will go.

IMPORTANT!

Before each test you have to insert a new unused indicator strip into the helix test body. Let the helix test body dry before reuse. It is useful to blow the hose and the holding fixture of the indicator strip with compressed air. The helix test body should be renewed after 250 batches.

Use as working-day routine test of a Class B autoclave

1. Place the test body with the indicator strip inserted on a tray into the empty chamber.
2. Preheat the autoclave and start the **Universal-Program**.

Use as batch control system

1. Include the test body to the items to be sterilized and place it into the autoclave.
2. Then start the respective program (B-cycle, 134°C, regarding DIN EN 13060).

After taking the MELAcontrol out of the autoclave, unscrew the end cap and check the indicator strip. If it shows a complete colour change it can be assumed that the autoclave sterilizes successfully.

Evaluation

<p>A</p>  <p>MELAcontrol® For use in small steam sterilizers 134-137°C</p>	}	<p>The complete colour change or with a slight gradient to the middle shows a successful steam penetration.</p>
<p>B</p>  <p>MELAcontrol® For use in small steam sterilizers 134-137°C</p>		

HINT!

Depending on the manufacturing processes, the colour change can be different (from dark blue to light violet). The decisive factor for the assessment is the consistent and complete colour change of the indicator strip whereas a slight gradient to the middle is still acceptable.