Produktname

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Produktbeschreibung

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MELAfol reels and pouches

Product description

MELAfol reels and pouches are used to package medical devices for sterilization and maintain the sterility of the enclosed devices until they are used.

MELAfol reels and pouches for sterilization comply with ISO 11607-1 and EN 868-5. They are made of PET/PP copolymer films and medical grade paper. There are process indicators on the surface of the reels and pouches. After sterilization with steam, the process indicators show that the device has been sterilized by changing colour from pink to brown. **PLEASE NOTE:** The change in colour does not indicate whether the sterilization was successful.

MELAfol reels and pouches are marked with a LOT number. This number allows the device to be traced.

General notes

- ▶ The sealing temperature for MELAfol reels and pouches is 170-190 °C. MELAG recommends a sealing temperature of 180 °C.
- ▶ MELAfol reels and pouches are not suitable for reuse.
- ▶ Do not use damaged MELAfol reels and pouches.
- ▶ Do not use MELAfol reels and pouches that have become wet.
- ▶ Please observe the regulatory requirements for the storage period of sterile materials (in Germany e.g. DIN 58953, Part 8 or the DGSV guidelines).
- ▶ Please note that all serious accidents which occur in connection with the medical product (e.g. death or serious deterioration in the state of health of a patient) which were presumably caused by the product, must be reported to the manufacturer and the relevant authority of the member state, in which the user and/or patient resides.

Application

1. MELAfol pouches are ready for use. Cut MELAfol reels to the desired length. Seal the cut at one end to make a pouch.
2. Cover sharp ends of instruments with an instrument protector to prevent punctures and cuts.
3. Place the item to be sterilized in the pouch. Fill to a maximum of 75 % of the pouch volume.
4. Leave a minimum distance of 1.5 cm between the item to be sterilized and the sealing ends to ensure an even air flow during sterilization.
5. Before sealing, remove as much air as possible from the pouch.
6. Seal the pouch with a sealing device according to the manufacturer's recommendations.
7. Fill the pouch horizontally on a flat surface to avoid stressing the sealing seams.
8. When loading the steam sterilizer, make sure that either the film or paper sides of different pouches are facing each other.
9. Place the pouches on edge in baskets, containers or foil holders for optimal drying. Make sure there is enough space between the pouches to ensure an even air flow during sterilization.
10. Sterilize the MELAfol pouches.
11. Handle the pouches with care when removing them from the steam sterilizer to avoid cracking. Store the pouches in an area protected from sunlight, water and other liquids.
12. Do not use wet pouches. Repeat the sterilization process with a new pouch.
13. Hold the paper side to open the pouch. Slowly pull off the plastic side in the direction the peel arrow to get a clean peel result.

Storage instructions

Store MELAfol reels and pouches at max. + 35 °C and max. 70 % humidity and protected from excessive dust, air flow, humidity and direct sunlight. If these conditions are observed, unused MELAfol reels and pouches have a shelf life of 5 years from the date of manufacture.

Disposal

Dispose of used MELAfol pouches in the domestic waste. **PLEASE NOTE:** Label contaminated MELAfol pouches and dispose of them in the designated special waste (e.g. in biohazard bags).

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MELAG Medizintechnik GmbH & Co. KG