FR

KRUUSE Anaesthesia Induction Mask

Mode d'emploi

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Instruction for use

FEATURE

- Transparent housing for easy observation; soft cushion for leak-free use
- ISO 15mm OD connection (ISO-5356-1)
- Durable material (polycarbonate and silicone, autoclavable up to 121°C)

CAUTION

- Be sure to use correct size. The mask should cover the patients nose and mouth, providing an airtight seal. It is important to choose the correct size to ensure proper fit, avoiding leakage, or creating unintended pressure causing reduction in venous return and edema of the muzzle. Dead space should be minimised by choosing correct mask shape and size
- It often requires sedation to tolerate induction masks
- Masks do NOT protect airways from aspiration of foreign materials
- This product is reusable, please clean and disinfect at first time before using on the patient
- Do not use if the package is exposed to moisture, or if mold is identified on the mask or packaging
- Do not use after expiry date. If past the expiry date, please discard it
- Please check the appearance and function before use on the patient. If there is any defect observed on the cushion ring or housing, use of a new mask is suggested
- Storage: room temperature

The reusable anaesthesia mask should be cleaned and disinfected as follows:

- When first using the new anaesthesia mask
- Before use on a new patient
- Whenever the anaesthesia mask becomes contaminated
- Clean the masks with soap/disinfectant intended for silicone and polycarbonate material. Completely rinse off soap/disinfectant with water after cleaning and before use
- Every 24 hours

WARNING

It is recommended to use this mask for single patient use. By use between different patients, clean the mask, to avoid cross contamination. Reusing, soaking, rinsing or disinfecting this device with soap/disinfectant intended for silicone and polycarbonate material.

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