- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

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KaWe

Allzweck-Verband-Schere

| | | |
|-----------|--|----|
| de | Gebrauchsanweisung Allzweck-Verband-Schere | 4 |
| en | User's Manual All-purpose bandage scissors | 7 |
| fr | Mode d'emploi Ciseaux universels pour pansements | 11 |
| it | Istruzioni per l'uso Forbici multiuso per bendaggi | 15 |
| es | Instrucciones de empleo Tijeras universales para vendajes | 19 |
| pt | Manual de operação Tesouras para ligaduras | 23 |
| ru | Руководство по применению Универсальные ножницы для перевязок | 27 |

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User's Manual Universal bandage scissors

Dear Customer, thank you for choosing a KaWe product. Our products are known for high quality and durability. This KaWe device complies with the provisions of Regulation EU 2017/745 (European Regulation on Medical Devices), whereby it is classified as a medical product of class I.

Please carefully consider this instruction and read all the provisions prior to use. Follow the given care instructions.

Application: Only authorized qualified personnel may use the instruments.

Intended use: For cutting bandages and dissecting garments. Field: general medicine, e.g., medical practices.

Target patient group: All

Improper use/contraindications to the use of the product: Any other use or use outside the above intended use shall be considered improper. The manufacturer is not responsible for damage resulting from such use. In this case, all risks are assumed solely by the user.

Warnings:

Secondary treatment limitations: Frequent secondary treatment has little effect on these instruments. The service life of the product is usually defined based on the degree of wear and use-related damage. Ultrasonic cleaning is not permitted! High-speed autoclaving and hot air sterilization will damage the instrument!

Additional information: If used and stored properly, these KaWe products will serve you reliably for many years.

Instructions:

en

Preparation for cleaning: After using the scissors, we recommend immediately rinsing them under running water or in a slightly alkaline solution to avoid possible drying of residues after use (e.g., blood). In order to properly carry out steam sterilization (see “Cleaning, Sterilization” section), it is always necessary to carry out a thorough pre-cleaning!

Manual cleaning: According to the recommendations of the Robert Koch Institute (RKI), the mechanical treatment is preferable. Manual cleaning is not recommended.

Automatic cleaning: Equipment:

CM 310 (Maquet), cleaning agent (neodisher®FA forte 0.4%/neodisher®Z 0.2%)

G7828 (Miele), cleaning agent (Mucapur®XL 0.4%/Mucapur®Z 0.15%)

WD 390 (Belimed), cleaning agent (Mucapur®AF 0.5 %/Mucapur®Z 0.1 %)

1. Thoroughly wash the instruments under running water before mechanical treatment to avoid the remnants of cleaning/disinfecting agents entering the machine.
2. Place the instruments in a special container.
3. Place the instrument container in the cleaning and disinfection device so that the water jets directly fall on the instruments.
4. Follow the instructions for the detergent dosage.
5. Start the Vario TD program followed by thermal disinfection. Thermal disinfection is provided in accordance with the established value of AO (the disinfection time, in seconds, at a certain temperature) and in accordance with national regulations (EN ISO 15883).
6. When the program is completed, remove the instruments from the cleaning and disinfection device and dry them (the Robert Koch Institute (RKI) recommends to dry with compressed air). Special attention should be paid to the places of the instrument container that are difficult to reach during drying.
7. Visual inspection for integrity and cleanliness with a suitable magnification object (based on experience, 8x magnification is sufficient for visual inspection). If there are residues of contamination on the instruments after mechanical treatment, repeat cleaning and disinfection until they are completely cleaned.
8. Attention! Only mechanical cleaning (without confirmed disinfection) requires subsequent thermal disinfection in a steam sterilizer placed in suitable trays or mesh containers.

Disinfection: For example, Heliapur can be used to disinfect the instruments. The appropriate concentrations can be found in the manufacturer's instructions.

Sterilization: The instruments can be successfully sterilized with the following parameters: Fractional pre-vacuum (three times), sterilization temperature: 134 °C, exposure time: at least 3 min. (full cycle), drying time: 10 min.

Maintenance: Damaged parts shall be rejected or replaced. Surgical lubricant can be applied to moving parts, such as hinges.

Functional control and check: Check the ease of movement of the moving parts of the scissors. Scissors assembly: Perform a visual inspection for damage and wear.

Package: Individual: A standard plastic bag can be used. The instrument bag must be of sufficient size so that the clasp is not under tension.

Transportation and storage:

Keep the instrument away from dust, humidity and contamination during storage and transportation.

Temperature: -10 °C to +50 °C

Relative humidity: from 30% to 75%

Disposal: Scissors (stainless steel and plastic) must be handed over to a recycling center after prior preparation. Please observe the local regulations.

General accessories: For more information on these products, please visit our home page: www.kawemed.com.

Manufacturer: **KaWe**

Manufacturer's contact details: Dealer's address or phone number, for enquiries:
+49-7141-68188-0.

en

Basic UDI-DI: 4030155KaWe1103Z4

Symbols:

| | | | |
|---|------------------------------------|---|---------------------------------|
|  | Manufacturer & date of manufacture |  | Please follow the User's Manual |
|  | Medical device |  | CE conformity mark |
|  | Lot code |  | Attention! |
|  | Article number |  | UDI Data Carrier |

Warranty: The warranty totals two years from the date of purchase if used properly and in accordance with the User's Manual. If any issues arise or repair is necessary, please contact your dealer.

Information for users and patients

All serious incidents involving this device must be immediately reported to the manufacturer and the competent authorities of the respective Member State in which the user and/or patient is located.

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KIRCHNER & WILHELM GmbH + Co. KG